

# Arterial Line Trainer

## Materials:

1. Dragon Skin FX-Pro or Ecoflex-0030 (~\$15/model)
  2. Slacker (~\$3/model)
  3. Metamucil (~ 30c/model)
  4. Latex rubber tubing
  5. 3-way stop cock
  6. 20cc syringe
- Male Manequin Hand Jewelry Display (\$10, re-usable)
  - OR
  - 1/4inx8in steel rods (~\$5)
  - 3in x 8in PVC Pipe (~\$3)
  - scrap wood (\$?)

## Model How-To:

1. Equal parts A and B of dragon skin
2. 10% slacker by volume
3. 5-10% Metamucil by volume
4. Mix thoroughly and pour into mold
5. Remove from mold when cured and attach tubing

## Mold How-To:

1. Cut PVC pipe as shown in picture.
2. Cut 2 pieces of wood to act as end caps and glue them to 5inx5in scrap wood
3. Drill 1/4in holes in both wood end boards ~ 1cm from base of PVC pipe and ~ 3cm spaced apart
4. Slide steel rods through holes

